

Flex glue-on boots

Flex glue-on boots are a great option for horses that ride long distances, or if for any reason you need continuous hoof protection for a few days.



Flex glue-on boots are sold individually.

How to fit?

- The boot must fit snugly.
- Hoof must be dry, well-trimmed and clean.
- We do not recommend using Flex glue-on boots for more than 7 days at a time.

How to use:

- Stand the horse on a clean, even surface, with a downward slope.
- Trim the hooves carefully and rasp the hoof wall with the fine side of the rasp so you get a good surface for the glue to attach to.
- Clean the hoof wall and the inside wall of the Flex glue-on boot with rubbing alcohol. To avoid grease getting to the shell or hoof wall, do not handle them with bare hands after this.
- If you are gluing the boots on in very cold temperatures, you should heat up the Flex glue-on boot first, for example with a hair dryer.

Use Vettec Adhere two component glue to attach the Flex glue-on boots to the hooves. You will find Adhere from the nearest Vettec dealer.

- Add Adhere to the inside wall of the Flex glue-on.
- Lift the horse's foot up and make sure that the hoof is completely clean.
- Place the Flex glue-on boot on the hoof and tap it into place with a rubber hammer.
- Hold the foot up until the adhesive dries. Check the drying time guidance on the Vettec Adhere instructions.
- Then put the foot down and glue the holes and the top edge of the shell.
- Do not use Flex glue-on boots continuously for more than 14 days to prevent hoof overgrowth.

[Watch a video of how to attach the Flex glue-on boots.](#)

Flex Glue-on Boot sizes:

Boot size	width mm	length mm
110	105-115	110-120
120	115-125	120-130
130	125-135	130-140
140	135-145	140-150